

Spirit™ Family [Liquid Oxygen Portables] CAIRE® A Chart Industries Company

Patient Guide



How long will a Spirit™ last, and how light is it?

- Spirit™ 300: 7–9 hours at 2 LPM; 4.3 lbs. full
- Spirit™ 600: 13–17 hours at 2 LPM; 5.6 lbs. full
- Spirit™ 1200: 28–34 hours at 2 LPM; 8 lbs. full

How can it last so long? My current portable weighs 8 pounds and lasts the same duration?

- Spirit™ uses a revolutionary electronic sensing system that only delivers oxygen when you are breathing in. This conserves the supply and means that a small amount of liquid will last a very long time.

If Spirit™ will last so long, how can I be sure that I am receiving the correct dose?

- The oxygenation saturation achieved by Spirit™ is equivalent to all other types of oxygen therapy, including other ultra-portable liquid systems using conservers.

How easy is Spirit™ to fill?

- A simple one-step-operation is completed in under one minute, with no power required.

Will using Spirit™ increase my electricity bill?

- No, neither the base unit from which the Spirit™ is filled, nor the Spirit™ itself draws any power from your domestic electricity supply. The system also does not produce any excess heat or noise.

Do I need any proprietary accessories?

- No, the system is supplied complete, although some upgrade options are available to purchase. The system is delivered with everything you need to take to the road.

Will the Spirit™ be paid for by my insurance?

- Your physician will complete the paperwork necessary for you to make a claim to your health care provider. In general, if your doctor has decided that oxygen therapy is required, your Home Medical Equipment (HME) provider can supply any modality which will provide the prescribed dose. The best way to proceed is to sit down and talk to your HME who will discuss all of the options open to you, and together you can choose the therapy solution most appropriate for your lifestyle.

What do I do now?

- Contact your HME. The number is on the reverse side of this information

Features

- [Side fill or top fill compatible]
- [Compatible with all base reservoirs including other brands]
- [Back-up continuous flow (CF) setting]
- [Optimizes saturation levels through the consistent oxygen delivery rate]

Patient Benefits

- [Ultra light weight]
- [Single tube cannula]
- [Wide range of available accessories & carrying options]

Spirit™ Family [Liquid Oxygen Portables] CAIRE® A Chart Industries Company

Patient Guide

Now that you've been prescribed oxygen therapy to treat your respiratory condition, it is important to ensure that the therapy you receive supports the continuation of the lifestyle you desire.

For years, respiratory therapists have agreed that patients who remain active, spend time outdoors, and continue to live a healthy lifestyle will achieve the best long-term outcome. This has been difficult in the past as oxygen equipment has

either been heavy and bulky, or required frequent refilling, limiting the range and time from home base.

CAIRE, Inc.'s Spirit™ liquid oxygen portables have been designed with your freedom in mind. Our ultra-portable devices are offered in a soft-sided case, hard shell, or the new leather finish style. The Spirit™ will provide the average oxygen patient with therapy on the move for longer durations while maintaining its light weight.



CAIRE, Inc.
Spirit™ Portables are
available in YOUR area.

is a CAIRE, Inc. partner and will be
delighted to answer any questions regarding
Spirit™ units!

Chart Industries, Inc.
CAIRE, Inc.
2200 Airport Industrial Drive, Suite 500
Ball Ground, GA 30107
Ph 770-721-7759 • Toll Free 1-800-482-2473
Fax 770-721-7758



www.cairemedical.com

Chart BioMedical, Ltd.
Unit 2, Maxdata Centre
Downmill Road, Bracknell
Berks RG12 1QS, United Kingdom
Ph +44(0)-1344-403100 Fax +44(0)-1344-429224

Copyright © 2008 Chart Industries Ref 11855637 Rev F